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### ■ Navigation of the drill guide with Brainlab

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#### Attaching the reference star for use with Brainlab

The button on the reference star for use with Brainlab (CS 3929-06) must be held down in order to be able to mount the reference star on the drill guide (CS 3929-05). The reference star must be fully mounted on the adapter on the drill guide. The reference star must be mounted in a properly oriented fashion so that the pin on the drill guide engages in the corresponding slots of the reference star and the locking mechanism of the button snaps into place in the corresponding groove of the drill guide. Then the depth stop (CS 3929-02) can be pushed into the drill guide.

After the Brainlab Disposable Reflective Marker Spheres have been screwed on, the instrument is ready for manual calibration. Using the Brainlab Instrument Calibration Matrix (ICM4) and the associated insert for the Brainlab ICM4 (CS 3929-04), the instrument can be easily calibrated. The reference star must be mounted and oriented relative to the drill guide handle such that during use, the navigation camera has a direct and unimpacted view of the reference star.

CS 3929-06

CS 3929-05

CS 3929-02

CS 3929-01

CS 3929-04

CS 3928-04