


SPASY™

Bone Cement Dispensing Kit


Indication

- VCF (Vertebral compression fracture) due to Osteoporosis
- Osteolytic fracture
- Metastatic bone fractures



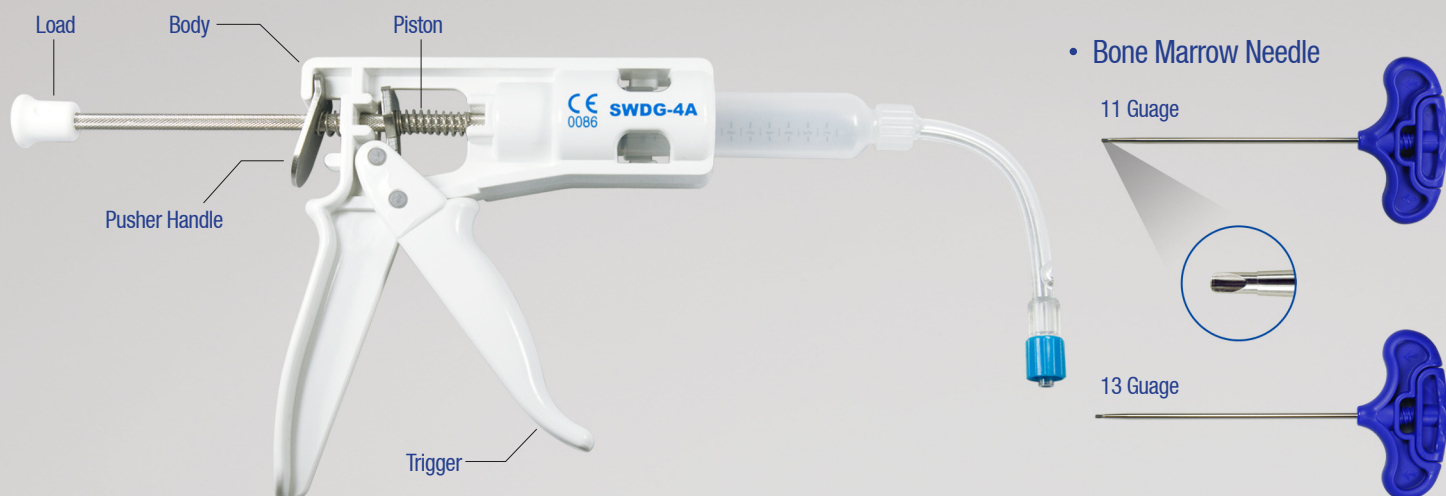


Step-1

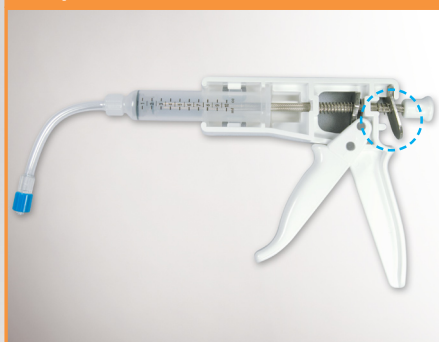


Push the pusher plate and pull out the load to fix the piston in the middle of the body.

Push the Pusher Handle

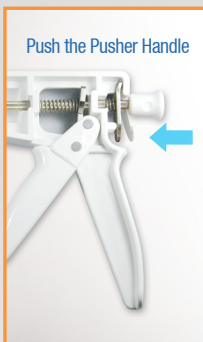

Pull out the Load

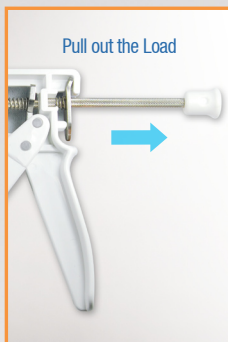

Step-2



Pour the cement carefully into the syringe leaning to the left side as presented above

Step-3



Turn the syringe to the right till the "Lock sound" is audible. Then, pull the trigger to remove the air inside of the syringe.

Step-4



Assemble the hose to the Bone Marrow Needle. Pull the trigger with the handle slowly to press the piston in the syringe to push the bone cement.

Ordering Information

| Part No | Description | EA/Kit |
|------------|----------------------------|------------|
| SWDG-4A | Bone Cement Dispensing Kit | 1EA |
| SWVP-11A | Bone Marrow Needle | 2EA |
| SWVP-13A | | |
| Article No | Gauge | Length(mm) |
| SWVP-11A | 11 | 125 |
| SWVP-13A | 13 | 125 |