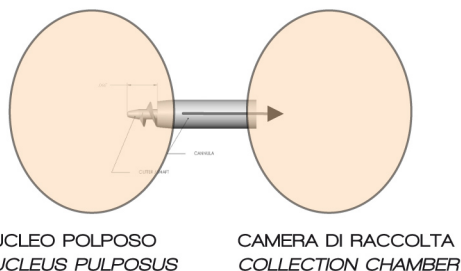
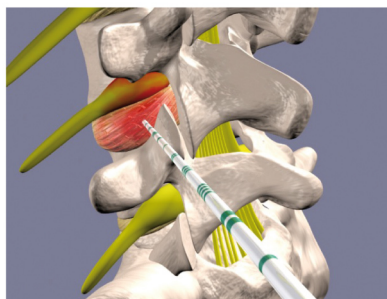
DISCECTOMIA PERCUTANEA
PERCUTANEOUS DISCECTOMY

Tecnica Operatoria – Surgical Technique

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Percutaneous discectomy is a minimally invasive procedure that use the Archimedes's principle of the screw pump, in order to remove the nucleus pulposus in case of herniated disc, reducing the pressure in the disc and area surrounding the painful nerve root. The procedure is suitable for any level of the spine with good advantages for the patient in term of anesthesia, recovery time, and pain. The access is made by a needle skin puncture with no cicatrix and no suture: the most part of patients go home the same day of the treatment. The procedure is performed under fluoroscopic guidance. By removing the nucleus from the herniated disc the patient feels an immediate result, in minutes or few hours from the procedure. The quantity of material removed is from 0,5 cc to a maximum of 2 cc and the entire procedure can take form 10 minutes from maximum 1 hour.

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
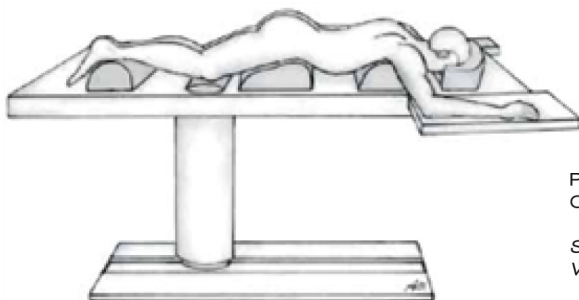
Is very important the way to position the patient on the table, in order to reduce any load on the fractured vertebral body.

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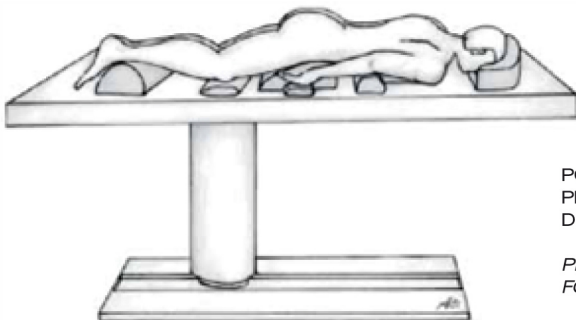*STANDARD PRONE POSITION
WITH ARMS EXTENDED*

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*PRONE POSITION WITH ARMS ADDUCTED
FOR UPPER THORACIC APPROACHES*

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The procedure can be operated under local or general anesthesia.

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Is important to determinate the best inclination for disc access before starting the procedure, and the inclination of the needle is very important to reach the nucleus pulposus of the disc without compromising any other organs.

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Under continuos fluoroscope detection insert the needle since reach the anulus of the disc and verify under the lateral view the correct inclination and direction of the needle.

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Insert the needle inside the disk till reach the nucleus pulposus, and remove the stylet from the needle leaving the cannula located inside the disk.

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Insert the cochlea's device inside the cannula. Fix the device to the cannula rotating to the right.

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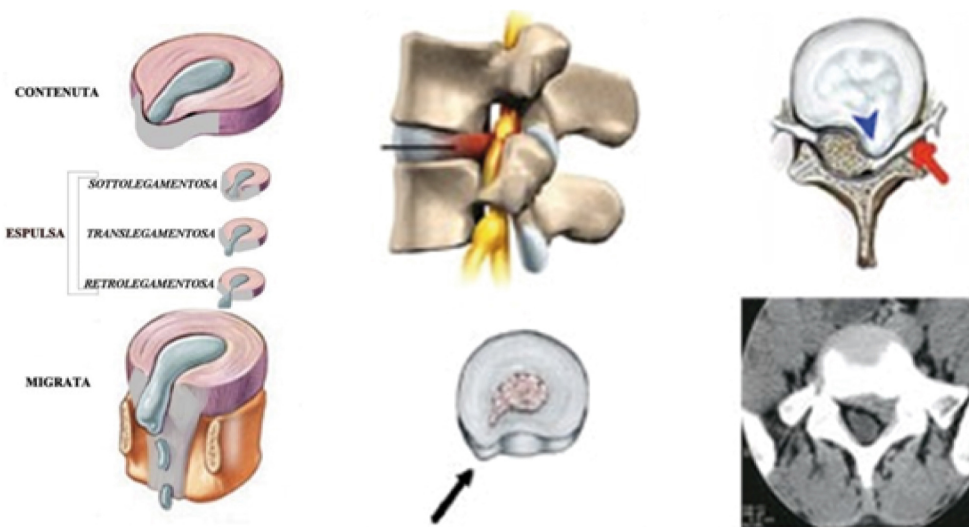
Turn on the device pushing the on / of button. Start to move up and down slowly and carefully the device inside the disc under continuous fluoroscope monitoring. After few second (15/20 sec') of aspiration continuously moving up and down the device, turn off the engine and disconnect the device from the cannula. Remove the nucleus pulposus tissue from the cochlea using the annex spatula or sterile surgical towel. Reconnect the device to the cannula and repeat the operation till the quantity of the disk removed is enough to reduce the hernia. Finally the result is a discectomy of the nucleus pulposus or partial discectomy. Remove the needle from the disk and put a plaster on the needle hole.

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Discectomy procedures demonstrate success of nucleus removal and relieves pain in over 70% of cases.

- *Symptom recurrence rates post discectomy are low;*
- *Extrapedicular approach minimizes permanent structural changes;*
- *Avoids entry into spinal canal;*
- *Laminectomy not required;*
- *Percutaneous approach may lower surgical complication rates;*
- *Amount of nucleus pulposus removal needed is dependent from patient;*
- *Small changes in disc volume can result in a large changes in disc pressure;*
- *Surgeons rarely remove more than 30% of the nucleus pulposus;*
- *Successful outcome of disc surgery reported to be independent of the amount of disc material removed;*
- *Significant pain relief;*
- *Reduction of analgesic use;*
- *Quick recovery time, no overnight hospital stay*

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