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Fracture Stabilization

Based on the amount of contrast agent dispensed in the balloon that was read, prepare a corresponding amount of bone cement applicators (1 bone cement applicator = 1.5 mL cement).

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The injection must be carried out under continuous radioscopic control and must be done very slowly, verifying the exit of the cement from the cannula and its distribution inside the vertebral body. Possible extravertebral diffusion should be checked. If cement leaks from the vertebral body, immediately discontinue the application.

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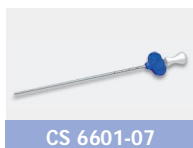
Timing for the Preparation and Application

The following times indicated were determined in the laboratory by performing tests under temperature-controlled conditions of 23°C ± 1°C and by using a 20-mL plastic syringe. Any increase in temperature of the working environment, the product or the preparation accessories over 23°C reduces the waiting and hardening times of the acrylic resin PMMA cement. Likewise, lower temperatures increase such times.

	OPERATION	DEFINITION	PHASE DURATION [sec]
1	Mixing	Mixing of the powder and the liquid	60
2	Application device filling	The mixture is transferred into the application system	60
3	Waiting	The cement cannot be used	300
4	Working	The cement can be applied	600
5	Hardening	The cement hardens, increasing its viscosity. Cement cannot be delivered anymore. Then the exothermic reaction takes place.	360

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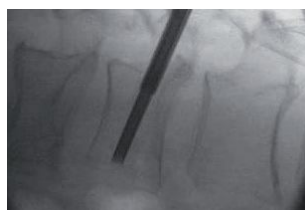
Filling the Bone Cement Applicator

Fill the bone cement applicators (CS 6601-07) with cement. To do this, attach the prepared, cement-filled Luer lock syringes to the bone cement applicators and inject the cement until at least 1 cm emerges from the bone cement applicators.

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
Remove the Luer lock syringes directly and introduce the stamp into the bone cement applicators. In doing so, extrude a small amount of cement from time to time and check the consistency. The optimal delivery consistency is reached when the cement is extruded like toothpaste from the bone cement applicators.

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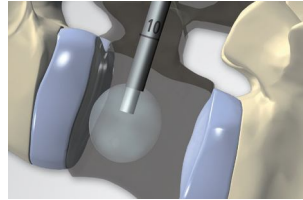The bone cement applicators (CS 6601-07) are now inserted through the cannula (CS 6601-03). Ensure that the bone cement applicators are positioned in the anterior portion of the cavity created in the vertebral body.

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Filling with bone cement

The bone cement is forced into the cavity using the stamp of the bone cement applicator (CS 6601-07). First leave the bone cement applicators in front and continuously pull back while dispensing cement. Monitor continuously under X-ray control. Dispense cement until the cavities created by the balloons are filled. The vertebral body must not be overfilled, particularly in the posterior area of the vertebral body. If cement should leak out of the vertebral body, immediately stop filling.

Note:
After filling the vertebral body with cement, introduce the empty cement applicator and leave in situ until hardening in order to avoid back flow of cement.

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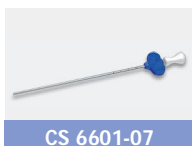
Remove all instruments

After completing cement filling, remove all instruments and close the stab incisions according to the standard technique.

CS 6601-07